



**EL-RAD. SERVICES LTD.**  
**Canadian X-Ray Supplies**

124 Garden Avenue  
 North Vancouver, BC V7P 3H2  
 Ph: 604.980.7400 Fx: 604.980.7510  
 www.canadianxraysupplies.com

## Veterinary Technique Chart

BODY PART	Grid	CM Size	FFD	MAS	KVP																			
					2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Extremity	No	40	3.3		48	50	52	54	56	58	60	62	64	66	68	70	72	74	76	78	80	82	84	86
Extremity	Yes	40	5		58	60	62	64	66	68	70	72	74	76	78	80	82	84	86	88	90	92	94	96
Shoulder	Yes	40	9		48	50	52	54	56	58	60	62	64	66	68	70	72	74	76	78	80	82	84	86
Schest / Ribs	Yes	40	5		52	54	56	58	60	62	64	66	68	70	72	74	76	78	80	82	84	86	88	90
Chest	Yes	40	2.5		80	82	84	86	88	90	92	94	96	98	100	102	104	106	108	110	112	114	116	118
Skull	Yes	40	5		58	60	62	64	66	68	70	72	76	78	80	82	84	88	90	92	94	98	100	102
C. Spine	Yes	40	5		64	66	68	70	72	74	76	78	80	82	84	86	88	90	92	94	96	98	100	102
Thoracic Spine	Yes	40	5		66	68	70	72	74	76	78	80	82	84	86	88	90	92	94	96	98	100	102	104
Lumbar Spine	Yes	40	5		66	68	70	72	74	76	78	80	82	84	86	88	90	92	94	96	98	100	102	104
Pelvis	Yes	40	5		66	68	70	72	74	76	78	80	82	84	86	88	90	92	94	96	98	100	102	104
Kidney / Bladder	Yes	40	2.5		82	84	86	88	90	92	94	96	98	100	102	104	106	108	110	112	114	116	118	120
Abdomen	Yes	40	5		64	66	68	70	72	74	76	78	80	82	84	86	88	90	92	94	96	98	100	102
Fluid Level	No	40	10		50	52	54	56	58	60	62	64	66	68	70	72	74	76	78	80	82	84	86	88
Fluid Level	Yes	40	8		64	66	68	70	72	74	76	78	80	82	84	86	88	90	92	94	96	98	100	102
Birds & Reptiles	No	40	1.8		50	52	54	56	58	60	62	64	66	68	70	72								

\*For High Frequency Generator Reduce MAS 25%-50%

\*These techniques are to be used as a **guide** only, it may vary according to your equipment calibration, film, screens and processing type

\*For more information please contact PJ Mohajer, [pj@canadianxraysupplies.com](mailto:pj@canadianxraysupplies.com) or Ph (604) 980-7400